

sightline


COMFORT YOU CAN SEE.

DONNING and DOFFING INSTRUCTIONS



DONNING

1. Insert blue threaded male connector of PAFLEXBT tube into blue female connector on hood so it is worn - angled downward - toward the EVA or EVAHL Blower. To secure in place, turn clockwise until hand tight.
2. With EVA PAPR blower unit running, put on Sightline Series hood. Pull the hood over your head until the hood is comfortable on your head.
3. Tuck inner bib (Front and Back) of hood into shirt or protective clothing.
4. Reach underneath the front outer bib to attach clip to shirt or protective clothing.
5. Tighten drawstring neck cuff by pulling the sliding lock toward the chin until lock is against the Tychem material and the hood is secure around neck.
6. Pull the outer bib down over the outside of clothing and adjust front and back bib so they lay flat against your protective clothing.
7. Confirm breathing tube is not twisted or pinched after donning.

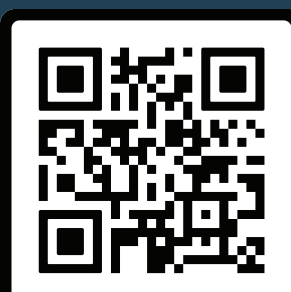

SCAN ME



DOFFING

1. Before doffing the hood, the user must be in a safe, hazard-free area and do the following (in conjunction with your employer's standard operating procedures).
2. Reach under the front outer bib to open the clip and loosen the drawstring.
3. With hood off, turn the EVA blower unit off by holding down the ON/OFF switch for 5 seconds. This is confirmed by a long beep and a shutdown of the EVA blower motor.
4. Clean and inspect components, as necessary, in conjunction with your employer's standard operating procedures, before storage.